

Implementing the ImPACT Program

Doug Tauchen

Director of Technical Support
ImPACT Applications

Iowa City, IA

Phone: (800) 942 -8632

doug@impacttest.com

Implementing the ImPACT Program

- ImPACT purchase, setup and registration
- Group baseline testing recommendations
- Online ImPACT walk through
- Questions

What's the Goal?

- Need for Concussion Protocol

- Neuro-Cognitive Testing



- Interpreting ImPACT Test Results

- Clinical ImPACT Case Work

Overview

- Online / Desktop
- Getting started with ImPACT Online
 - Computer requirements
 - Lab testing (Tech and ATC perspective)
- Navigation of ImPACT
 - Access of data
 - Printing and sending reports
- Common issues / Support

Online / Desktop choice

	Online	Desktop
Internet connection needed	Yes	No*
Data Storage	ImPACT Secure Server	Local PC or School Server
Data back-up	ImPACT	User
Meets HIPAA standards	Yes	Yes
Multiple workstation set-up	Yes	Yes
Support by email	Yes	Yes
Support by phone	No	Yes
Supports Sideline ImPACT	No	Yes
Ability to print reports	Yes	Yes
Macintosh compatibility	Yes	No

What would I suggest?

- Take a Demo
- Talk with your Tech. team
- Talk with other clinics/schools

Switching to Online ImPACT

- Data doesn't transfer between systems
- Run both programs until athletes cycle out of Desktop ImPACT
- Retest athletes in Online IMPACT.

Online Computer Requirements


- www.impacttestonline.com/results

Sign In

Login:

Password:

Sign In

 Data encrypted with 128-bit keys.

Where to find your log in information



+ Our system has automatically sent you an e-mail with your user name and password when we registered you

+ If for some reason you can't find the e-mail we sent you, please contact us at support@impacttestonline.com

Forgot Your Password?

Enter your user name and we'll e-mail you your password.

Retrieve Password

[Tech Facts](#)

[Support](#)

[Privacy Policy](#)

[Feedback](#)

Online Computer Software Requirements

- Adobe Flash Player (version 8 or newer)
 - Free download – www.adobe.com
- No proxy server
- Pop-up blocker off

Take the online demo at:

www.impacttestonline.com/impactdemo

Getting Started with Online ImPACT

- Send in the order form:
www.impacttestoffice.com
- You will receive by email 3 forms
 - ASP Agreement
 - Registration Form
 - Authorized User Form

Support Expectations

- 48 hour (2 business days) turn around on start up.

Spam/Email filters

Attachment filters

Fax issues

Online Test Site: Baselines

- Baseline testing

www.impacttestonline.com/????

No data or report access.

No login required

Open link

Lab testing

Web link


Online Test Site: Post Injury

- Follow up testing / Viewing reports
www.impacttestonline.com/results
(password protected) – Don't give out.


Sign In

Login:

Password:

 Data encrypted with 128-bit keys.

Where to find your log in information



- + Our system has automatically sent you an e-mail with your user name and password when we registered you
- + If for some reason you can't find the e-mail we sent you, please contact us at support@impacttestonline.com

Forgot Your Password?

Enter your user name and we'll e-mail you your password.

Tech Facts Support Privacy Policy Feedback

Group Baseline Testing Recommendations

Computer Lab Baseline Testing

- Test Lab before going live
- EXTERNAL MOUSE (invalid without)
- Control the testing environment
 - Empty terminal between athletes
 - 10-15 athletes per administrator
 - Lower the lights
- Use the baseline testing link
- Type the link when you are ready to start

Computer Lab Baseline Testing

What is required for baseline testing.

- Baseline testing link
- Name / Date of birth
- No SS# required

Who can supervise / proctor the exam?

- Taken test twice themselves
- Understand the basic process
- Interpretation is done by MD, or PhD.

ATC Recommendations

- 45 minute blocks (first year).
- Explain the test
 - Not an intelligence test
 - Want to complete as fast and accurately as possible
 - Difficult - not going to get everything right
 - Read direction twice before starting test – raise hand with questions
- Take demographic section as a group
 - Type the link together
 - Symptom inventory, start at own pace.

ATC Recommendations cont.

- Educate staff / parents
 - Have all the coaches test first
 - Have coach present for testing
- Eliminate the mad rush
 - End of school year testing

Baseline Testing

- Minimum interval between baseline tests is 2 years (until the age 19).
- Review athletes data to determine validity of baseline test.
 - Repeat invalid baseline tests
 - If invalid test is missed, simply use normative data during post-concussion evaluation

Common Errors/Fixes

- Forgot password
 - Lost password box on results site
- Buttons on webpage don't work
 - Update Adobe Flash Player
- "Error generating report"
 - Logout and back in / Restart
- Can't locate test just taken
 - Logout and back in / Restart
- Name listed twice
 - Different spelling or date of birth
- "Saving Data" screen remains on

Support / Feedback

- 1.800.942.8632 or doug@impacttest.com
- Technical / Program features or function
- What is going to make the program better for you?